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
## Electronic Bodytoner BT 850 with app control

Instruction manual - *Please read carefully!*

### GB Device and controls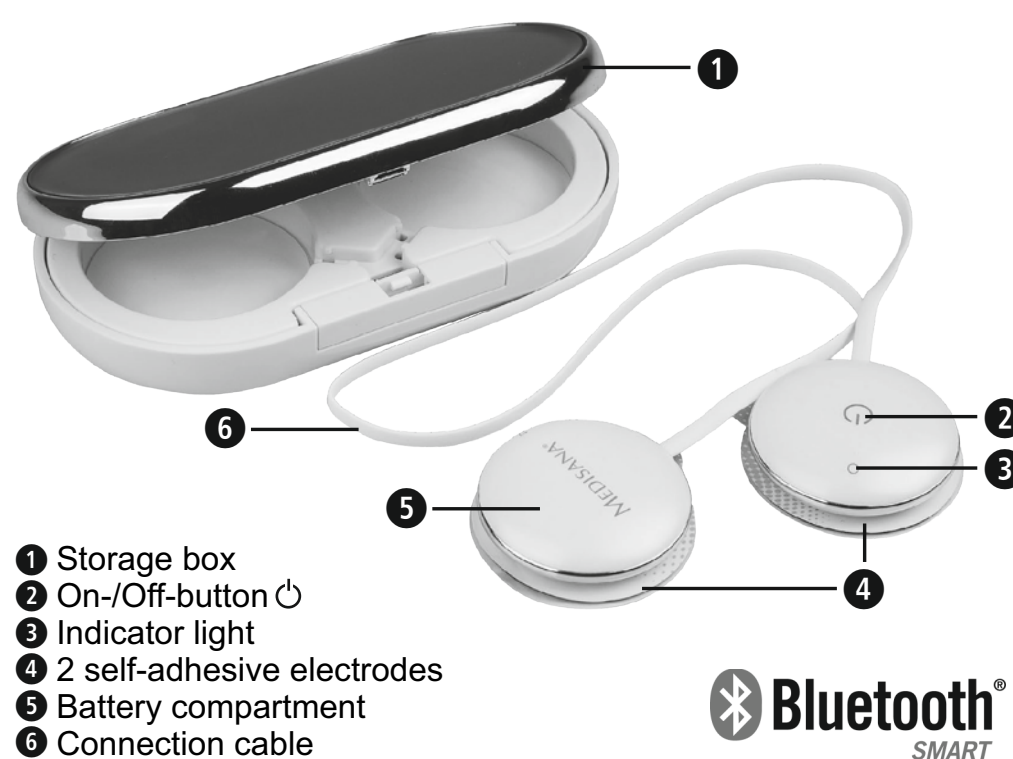

- 1 Storage box
- 2 On-/Off-button
- 3 Indicator light
- 4 2 self-adhesive electrodes
- 5 Battery compartment
- 6 Connection cable

### Designated use

This device is assigned for external usage on adults. It is a product designed especially for abdomen muscle stimulation with the following effects:

- Increase of blood flow in the treatment area
- Relaxation of muscle spasm
- Muscle building

### The device must not be used:

- by children without supervision
- in case of swellings, inflammations, burns, injuries, eruptions or sensitive points in the area of use (e.g. varicose veins, etc.)
- in case of implanted pacemakers or other implants or prosthetic materials in the body
- in case of cancer
- in case of irregularities of the heart or heart diseases (e.g. arrhythmia or myocardial defects)
- in case of epilepsy
- in case of serious arterial circulatory problems or when there is an increased tendency to bleed
- in case of abdominal or inguinal hernia
- in case of pregnancy resp. labour pain
- in case of acute illnesses and/or fever
- in case of pain of unknown origin

### Explanation of symbols

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

#### WARNING

These warning notes must be observed to prevent any injury to the user.

#### ATTENTION

These notes must be observed to prevent any damage to the device.

#### NOTE


These notes give you useful additional information on the installation or operation.

LOT number

Manufacturer

## GB Safety Instructions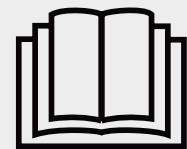

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Only use this device for the purpose for which it was developed and in the manner specified in the instructions for use.
- When using it for other purposes, the warranty expires.
- Do not use accessories that are not recommended by the manufacturer.
- Avoid the contact with pointed or sharp objects.
- This device is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- In case of a malfunction, do not try to repair the device yourself. Not only does the warranty expire, but serious danger can result (fire, electric shock, injuries). Only authorised service offices should carry out a repair.
- Never use the device while taking a bath or shower. Should liquids penetrate the device, immediately remove the battery and avoid further application. Contact your specialist shop or your service centre.
- In case of diabetes or other diseases you should consult your family doctor before applying the device.
- Persons, whose sensitivity is disturbed or impaired may only apply the device according to the instructions of a physician.
- The application should be pleasant. If you feel pain or feel the application unpleasant, stop it and consult your doctor.
- Ask your doctor for each kind of pain or disease, before using the device.
- Do not apply the device during the operation of a machine or while driving a car.
- If the device shows obvious damages at the housing, the cable or the electrodes, send it to your specialist shop or directly to the MEDISANA service center for repair.
- The device is only for private use and is not intended for medical or commercial purposes.
- Do not use the unit near to equipment emitting a strong electric or magnetic field, such as radios, microwaves or mobile phones. This may impair the correct functioning of the unit.
- Do not use the unit near any heat sources, highly flammable substances, gases or explosives.
- If you wear jewellery or piercings make sure that you take it off before using the unit. Otherwise it may cause punctual burnings.
- Do not use this device simultaneously with other devices which give electrical pulses to your body (e.g. TENS devices).

### Information about the electrodes

- Do not place the electrodes to open skin parts.
- The unit must not come in contact with moisture. Take off the electrodes before taking a shower / bath or swimming.

- Do not use this device directly after taking a shower or a bath or after a workout.
- Make sure, that the skin parts are completely dry and clean before applying the electrodes.
- In case of skin irritation, stop the treatment and consult a physician.

### BATTERY SAFETY INFORMATION

- Do not disassemble batteries!
- Remove discharged batteries from the device immediately since they may leak and cause damage to the unit.
- Increased risk of leakage, avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed seek medical attention immediately!
- Insert the battery correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period!
- Keep the device & batteries out of children's reach!
- Do not attempt to recharge batteries! **There is a danger of explosion!**
- Do not short circuit! **There is a danger of explosion!**
- Do not throw into a fire! **There is a danger of explosion!**
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

### Scope of supply

Please check first of all that the unit is complete.

The following parts are included as standard:

- 1 MEDISANA Electronic Bodytoner BT 850
- 1 Battery (type CR2032) 3V
- 1 Storage box
- 1 Instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.


#### WARNING

Please ensure that the polythene packing is kept away from the reach of children!  
Risk of suffocation!


### Inserting / changing battery

Open the lid of the battery compartment 5 by lifting the lid carefully on the marked („Δ“) area. If you use the device for the first time, you need to remove the insulating chip. Insert the new battery (type CR 2032, 3V) with the „+“-side on top. Close the lid of the battery compartment.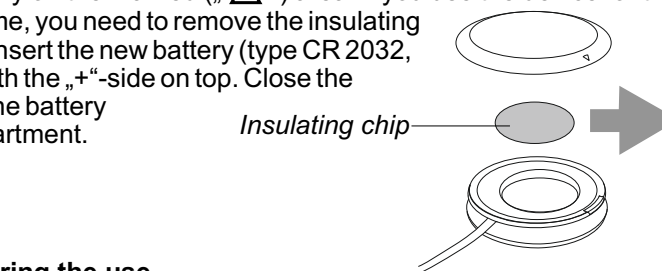

### Preparing the use

In order to use the device properly, you need to attach the electrode glue sheets onto the two electrodes as shown below before first use. The re-usable protection foil may be pulled off directly before usage and then the glue sheets can directly be attached to your skin.

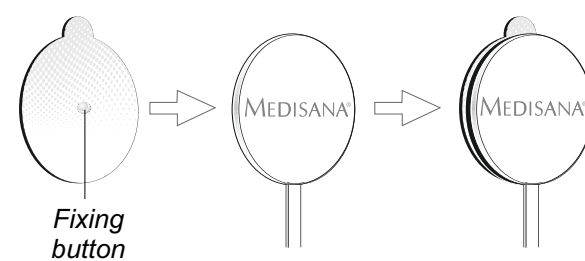### Downloading the app

Download the app and install it on your smartphone. You may download the app for free from below links or by scanning the following QR codes:

#### For iOS:

<http://www.medisana.de/bodytoner-ios-app>

#### For Android:

<http://www.medisana.de/bodytoner-android-app>

for iOS


for Android

### Connecting the app

Activate **Bluetooth** on your iOS- or Android device and start the app. Follow the instructions given by the app to pair your smartphone and the **MEDISANA Bodytoner** (only upon first installation) resp. to set-up or choose a personal account. The device will be activated by pressing the on-/off-button 2 one time. It will search for an active **Bluetooth** connection automatically, what will be indicated by a blue flashing indicator light 3. You can use the app completely after successful pairing.

### Starting up

In the main menu of the app you can select the desired program mode (Shoulders & Neck, Back, Waist, Limb). After selection of the mode you may choose other parameters in the following menu (e.g. the kind of application effect like pressing or tapping massage, the duration of application [from 1 to 15 minutes] and the intensity, which can be set in 7 steps). You may also call up the statistics of completed applications here. Now apply the electrodes as shown below by pressing the adhesive part firmly on your skin, so that the electrodes stick properly:

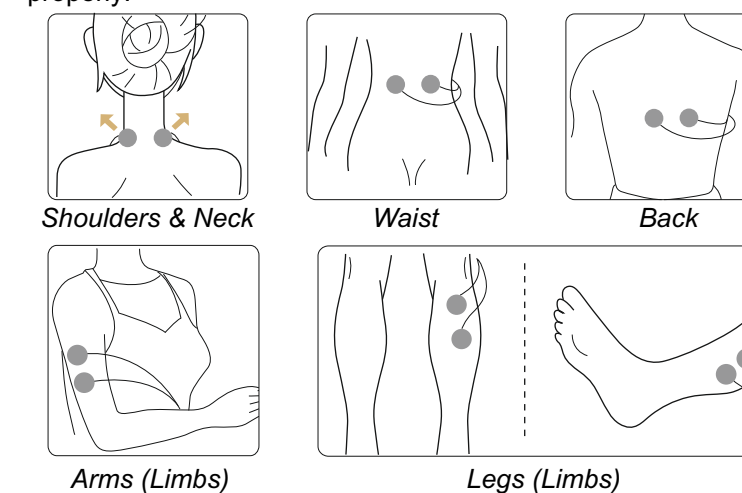Tap on „▶“ to start the massage. After the set duration time (or after max. 15 minutes) the device will automatically switch itself off. If you want to interrupt the usage before this time, tap on „■“ in the app.

Tap on „○“ to get more information regarding the use of the device in the selected area of your body. Tap on „<“ to return to previous menu.

#### WARNING

- If you feel uncomfortable during the use of the unit, set a lower intensity. If it is still feeling uncomfortable or painful or if the skin is itching or aching, immediately stop the usage and consult a doctor.
- In the beginning, only set the intensity to level 1 or 2. Only later - when you got used to the usage of the body-toner - you may set higher intensities (from level 3 to max. 7).

### The function modes

You may call up two different function modes by pressing the on-/off-button 2. By pressing the button once you will switch on the device, the indicator light 3 will flash in blue until a **Bluetooth** connection with your smartphone has been found (the smartphone therefore has to be switched on and **Bluetooth** has to be activated). If the connection has been established successfully, the flashing blue light stops and the device can be used via the app as described before.

By pressing the on-/off-button 2 two times quickly and consecutively, the device will switch to automatic mode. The indicator light 3 will now flash with a higher frequency and the device starts to massage with changing program modes on intensity level 2 and a preset target time of 15 minutes. You may pause the massage by tapping the 4 button, which also transmits the control back to the app.

### Switching off the device

Press and hold the on-/off-button 2 for at least approx. 3 seconds to switch off the device.

#### WARNING

Restrict the usage of the MEDISANA Bodytoner to a maximum of 2 times daily with a duration for each application of max. 15 minutes.

### Cleaning and maintenance

- Keep the adhesive surface of the electrodes clean and avoid contact with dust, liquids, sharp objects etc.. The re-usable protection foil should be put back onto the adhesive surfaces after each use.
- Clean the unit from time to time with a soft, dry cloth. The adhesive surfaces may only be cleaned by holding them under smoothly running, lukewarm tap water. Use your finger to

slightly wipe the adhesive side to get rid off the remaining water. Do not use towels to clean or dry the adhesive surfaces in order to keep the glueing ability. The adhesive surfaces need to air-dry before you can use them again.

- Never use strong cleaning agents, very hot water, alcohol, naphtha, thinner or gasoline etc.
- Cleaning and user maintenance shall not be done by children without supervision.
- Never immerse the unit or any component in water. Be cautious not to get any moisture inside of the unit.
- Do not expose the unit to direct sunlight; protect it against dirt and moisture.
- Do not subject the unit to extreme hot or cold temperature.
- Keep the unit in the storage box when not in use.
- Store the unit in a clean, dry location.

### Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device/unit.

Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

### Directives / Norms

This device is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE". We hereby declare that the **Electronic Bodytoner BT 850, art. 88330** complies with the fundamental requirements of the European RTTE Directive 1999/5/EC. You can request the complete EC Conformity Declaration from Medisana AG, Jagenbergstrasse 19, 41468 Neuss, Germany, or you can also download it from the MEDISANA homepage.

### Technical specifications

Name and model:	MEDISANA Electronic Bodytoner BT 850 with app control
Power supply:	1 x battery CR2032 3V
Power consumption:	max. 0,1 W
Compatibility:	from <b>Bluetooth</b> 4.0
Pulse width:	< 300 µs
Frequency:	1 Hz - 1000 Hz
Output voltage:	0 - 100V (at 1 kilohm load)
Treatment time:	max. 15 minutes
Operating conditions:	0°C to 45°C; humidity 10% - 90%; Air pressure 700 - 1060 hPa
Storage conditions:	-10°C bis 70°C; humidity 10% - 90%; Air pressure 500 - 1060 hPa
Size of storage box:	approx. 12 (L) x 6,5 (W) x 2 (H) cm
Weight approx.:	24 g (including battery)
Article no.:	88330
EAN no.:	40 15588 88330 9
	Replacement adhesive pads (2 pcs. set), Article no.: 88334, EAN no.: 40 15588 88334 7

In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

### Warranty and repair terms

Please contact your supplier or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is 3 years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**MEDISANA AG**, Jagenbergstraße 19, 41468 NEUSS,  
Germany  
eMail: [info@medisana.de](mailto:info@medisana.de), Internet: [www.medisana.com](http://www.medisana.com)

### For service, accessories and spare parts, please contact:

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